

# Fitting & Care Instructions

**Model: DC-032**

Instructions on putting on brace:



Step 1: Open front flaps of the brace and place the brace behind your knee with tag on top



Step 2: Close the bottom flap securing the brace around your calf



Step 3: Close the top flap securing the brace around your thigh



Step 4: Pull bottom Velcro strap through D-ring and tighten to desired support



Step 5: Pull top strap through to desired tension and Velcro to opposite side



Step 6: To set flexion or extension remove the ROM hinge from inside the knee brace. Open top cover of the hinge and set the pins into the desired degree of flexion and extension.



**TREND MEDICAL**

## CARE INSTRUCTIONS

**Hand wash with mild detergent and air dry.**

*See reverse side for size adjustment*

## WARRANTY

**The DC-032 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.**

# Sizing Instructions

**Model: DC-032**



Open brace and leave flat on a table



Detach and separate the left and right pieces



Close or extend the right side over the left side to the measurement of the patients leg.



Velcro down the right piece on top of the left piece once the correct size is found



This brace can also be adjusted to patients with larger thighs and smaller calves. To do this place the right piece at an angle with the bottom tilted in.