

Fitting & Care Instructions

Model: DCT-0901-2



Instructions on putting on brace:



Step 1: Center brace on side of knee and feed strap 1 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 2: Feed strap 2 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 3: Feed strap 3 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 4: Feed strap 4 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 5: Once all straps are tightened, use provided wrench to turn dial counter clockwise. Do small 1/4 inch turns and walk to check relief. Repeat until the preferred amount of unloading is reached



Step 6: After initial fitting there are quick release snaps for easy removal and application on subsequent uses.

See reverse side for strap fitting adjustment instructions.

Strap Adjustment Instructions:



Step 1: After initial fitting if any of the straps were too long, lay the brace out open on a table



Step 2: Locate the straps that were initially too long. On the inside you can adjust the length of the strap

Adjust the strap length by pulling the strap toward the middle.

TREND™
M E D I C A L

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

WARRANTY

The DCT-0901-2 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.