

Fitting & Care Instructions

Model: DCT-002

Instructions on putting on brace:



Step 1: Open front flaps of the brace and place the brace behind your knee with tag on top



Step 2: Close the top flap securing the brace around your thigh



Step 3: Close the bottom flap securing the brace around your calf



Step 4: Pull bottom Velcro strap through D-ring and tighten to desired support



Step 5: Pull top strap through to desired tension and Velcro to opposite side



Step 6: To set flexion or extension remove the ROM hinge from inside the knee brace. Open top cover of the hinge and set the pins into the desired degree of flexion and extension.



CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

WARRANTY

The DCT-002 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.