

# Fitting & Care Instructions

Instructions on putting on brace:


**Model: DCT-70ABD**



Step 1: Place abduction pillow on hip of affected arm with the curved side against your body. Wrap the strap around your waist and connect in front.



Step 2: If the waist strap is too long; find the Velcro clamp at the end of the strip and remove

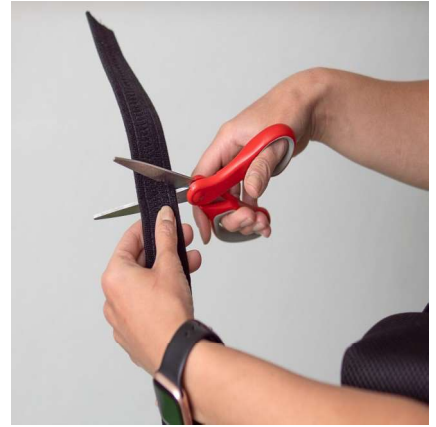

Step 3: Cut the strap to the desired length and reapply the Velcro clamp to the end




Step 4: Locate the sling portion of the brace and Velcro it to the side of the abduction pillow on your hip. Slide your affected arm into the sling. Bring the strap around your head and attach clip to the opposite side of the sling



Step 5: Find the strap attached to the clip and pull up to lift wrist to an inclined position. Velcro down once desired length is achieved. If the strap is too long, cut the strap to the desired length.



Step 6: Once product is fit it should look like the picture above.

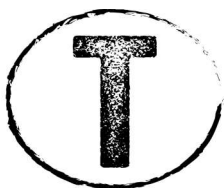

**TREND MEDICAL**

## CARE INSTRUCTIONS

**Hand wash with mild detergent and air dry.**

## WARRANTY

**The DCT-70ABD is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.**

