

Fitting & Care Instructions

Instructions on putting on brace:

Model: DCT-0901



Step 1: Center brace on side of knee and feed strap 1 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 2: Feed strap 2 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 3: Feed strap 3 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 4: Feed strap 4 through the D-Ring and tighten. If strap is too long refer to reverse side of this instruction sheet



Step 5: Once all straps are tightened, use provided wrench to turn dial counter clockwise. Do small 1/4 inch turns and walk to check relief. Repeat until the preferred amount of unload is reached



Step 6: After initial fitting there are quick release snaps for easy removal and application on subsequent uses.

See reverse side for strap fitting

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

WARRANTY


The DCT-0901 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.

Strap Adjustment Instructions:


Option 1:



Step 1: After initial fitting if any of the straps were too long, lay the brace out open on a table

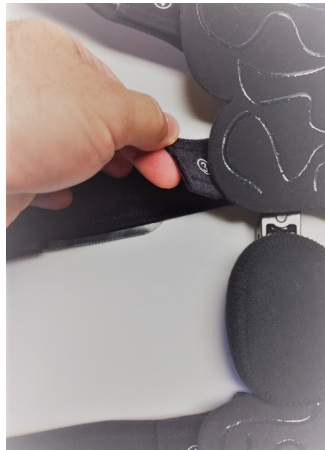

Step 2: Locate the straps that were initially too long. On the inside you can adjust the length of the strap

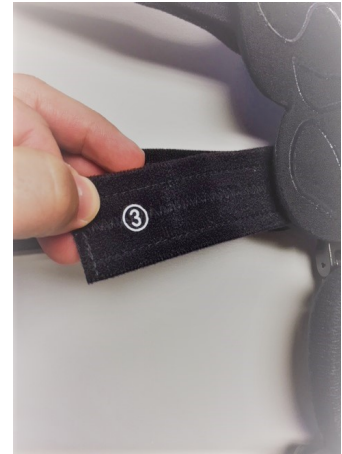

Step 3: Adjust the strap length by pulling the strap toward the middle.

Option 2:




Step 1: At the end of the strap that is too long is a numbered tab. Detach that tab from the strap



Step 2: Cut off the extra strap not needed.



Step 3: Reattach the numbered tab to the cut strap.

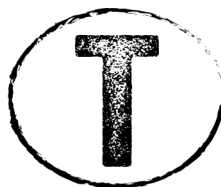

TREND MEDICAL