

Fitting & Care Instructions

Model: DCT-2397

Instructions on putting on brace:



When using the sleeve at the bottom of the brace: Place the wrap around the back of your leg. Pull and place the side with no Velcro down first, now pull the other side with the finger holes on top of itself snug and fasten it down with the Velcro.



When using the sleeve at the top of the brace: Place the wrap around the back of your leg. Pull and place the side with no Velcro down first, now pull the other side with the finger holes on top of itself snug and fasten it down with the Velcro.




When using the sleeve without the knee brace: Wrap the sleeve across the center of the knee and Velcro around to opposite side.



Pictured above is how the sleeve should look when using the sleeve without the brace.

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

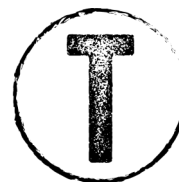

TREND MEDICAL

www.trend-med.com

WARRANTY

The DCT-2397 is covered by a 1 year warranty against all mechanical or manufacturer defects, from the date of purchase.