

Fitting & Care Instructions

Model: DCT-3233

Instructions on putting on brace:



Step 1: lay brace open on a table. Extend the hinges so they line up on the sides of the patients leg



Step 2: After measuring the patient, if the straps are too long, locate the Velcro clamps at the end of each strap



Step 3: Remove the Velcro clamp and shorten the length of the straps by cutting the strap



Step 4: Once correct size has been determined, remove white cloth in between the straps and velcro. Attach the straps against the Velcro strip so the straps do not move.



Step 5: To extend the length of the brace, locate the blue dials on the sides of the brace. Make sure to unlock both sides before extending. Once length is achieved, place both sides in locked position



Step 6: Completed view



TREND MEDICAL

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

WARRANTY

The DCT-3233 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.