

# Fitting & Care Instructions

**Model: DCT-3951**

Instructions on putting on brace:



Step 1: Remove lateral panels and lay brace on a table facing up. Find your size by referencing the sizing chart



Step 2: Remove overlapping front panel on both right and left sides.



Step 3: When completely detached your brace will look like the above image



Step 4: Find your size and attach to both right and left side.



Step 5: Once sizing is completed. Reattach the lateral panels and center the back panel



Step 6: Add higher front panel and center on abdomen. Overlap right side wing and attach to left side wing for a

## **WARRANTY**

The DCT-37 is covered by a 5 year warranty against all mechanical or manufacturer defects, from the date of purchase to original owner

## **CARE INSTRUCTIONS**

Hand wash with a mild soap and let air dry.



Step 7: Locate rubber pull tabs at the end of the right and left string and pull to desired support



Step 8: Once desired support is achieved attach the right pull tab and left pull tab

**TREND**  
M E D I C A L

Size 0	Size 1	Size 2	Size 3	Size 4	Size 5	Size 6	Size 7
22" - 32"	31" - 37"	36" - 40"	39" - 43"	42" - 46"	45" - 49"	48" - 52"	51" - 55"

Above measurement is based on waist circumference. There is an option for a 10" extension panel for patients larger than 55".