

Fitting & Care Instructions

Model: DCT-1012

Instructions on putting on brace:



Step 1: Open front flaps of the brace and place the brace behind your knee with tag on top



Step 2: Close the top flap securing the brace around your thigh



Step 3: Close the bottom flap securing the brace around your calf



Step 4: Pull bottom Velcro strap through D-ring and tighten to desired support



Step 5: Pull top strap through to desired tension and Velcro to opposite side



TREND MEDICAL

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

WARRANTY

The DCT-1012 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.