

# Fitting & Care Instructions

**Model: DCT-QF31**

Instructions on putting on brace:



Step 1: Open brace and lay flat on a table



Step 2: On right and left side lift and detach the overlapping sizing extension



Step 3: Find your size (SM-3XL) and reattach the overlapping sizing system to the correct size



Step 4: Repeat step 5 to the opposite side so both sides are the same size. Reattach the lateral panel to both sides



Step 5: Once properly sized center the back brace in the middle of your back (logo at the top) and wrap around your waist.




Step 6: Wrap the side wings right side overlapping the left side and pull the pull strings to your desired compression

## WARRANTY

**The DCT-QF31 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.**

## CARE INSTRUCTIONS

**Hand wash with mild detergent and air dry.**

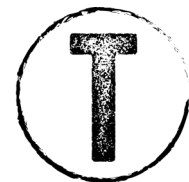

**TREND MEDICAL**